Lundatex® stocking

A compression stocking providing safe and precise pressure at all times.

The unique medical Lundatex® stocking provides safe, comfortable pressure at all times, whether at rest or when active, regardless of leg size/shape. The product ensures a precise pressure even after oedema reduction.

Intended purpose

The product is intended to be used as part of a treatment recommended by a medical professional, for the purposes of prevention, treatment and prophylaxis of multiple indications, such as phlebological and/or lymphological conditions.

Indications

- Venous insufficiency
- Oedema
- Lymphoedema
- Deep vein thrombosis
- Post-thrombotic syndrome
- Venous leg ulcers
- Mixed leg ulcers
- Varicose veins
- Sports injuries, such as blunt force trauma and sprains

Contraindications

If the following medical or physical conditions are present, advice from a medical professional is recommended prior to use:

- Peripheral arterial disease (ABPI < 0.8)
- Diabetes
- Presence of cardiac disease
- Severe skin infection
- Peripheral neuropathy
- Other contraindications for compression treatment

Instructions for use

It is recommended to put on your stockings first thing in the morning, or after lymph drainage. Your legs should be completely dry.

Instruction video

1. Grab the cuff using both hands, and pull the stocking over the foot and heel in one single motion. Do not fold or scrunch the shaft, as that will make putting the stocking on more difficult.
2. Pull the stocking up along the leg.
3. Use your hands to adjust and straighten out any creases or irregularities.

Warning/precautions

- Remove the product if discomfort or pain occurs. Contact your care provider if the pain is not eliminated or if it gets worse after removal.
- Never fold the cuff, as this will increase the pressure and may lead to dangerous constriction.
- Please ensure your footwear is in good condition, as defective shoe lining may damage the stocking.
- Note that sharp fingernails and/or jewellery can damage the material.
- Do not modify the product yourself in any way, as it might render the device ineffective or could even pose harm to your health.

Care instructions

The product is re-useable and washable. See the care symbols for instructions.

- Do not use bleaching agents, stain removers or softeners.
- Avoid using oily skin lotions. Wash after using lotions.
- Store the product in a dry place at room temperature, away from direct sunlight.

Usage and expected lifetime

- The product is intended as a single-patient, reusable product.
- The recommended length of use is six months, provided that proper handling is followed (usage, storage, care, etc.).

Disposal

When the product is ready for disposal, this should be done according to local specifications. The packaging and brochure are recyclable.

Report of serious incident

Due to legal regulations, you need to report any serious incident involving the use of this product to the manufacturer, and to your care provider and/or a responsible authority. See the manufacturer's contact details on this brochure and on the packaging.

Warranty

The legal regulations of the country of purchase apply. Please contact the retailer from whom you obtained the product, in the event of a potential claim under the warranty. Warranty may be limited or excluded if the handling and care instructions have not been followed. There will be no warranty if the product was not used according to the listed indications, if instructions given by a specialist were not observed, or if the product was modified by the user.

Symbols

	Expiration date
	Keep dry
	Consult the Instructions For Use
	Medical Device
	Do not dry clean
	Do not iron
	Dry in shade
	Keep away from sunlight
	Storage temperature limits
	Caution/precaution
	Does not contain natural latex
	Do not bleach
	Do not tumble dry
	Maximum temperature, gentle machine wash

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