## EN English *Lundatex*® medical / *Lundatex*® medical *cohesive*

*A compression bandage providing safe and precise pressure at all times.*

The unique Lundatex® medical / Lundatex® medical cohesive bandage provides safe, comfortable pressure at all times, whether at rest or when active, regardless of leg size/shape or applicer. The bandage includes a PressLock® fastening device which secures the bandage end. The product ensures a precise pressure even after oedema reduction.

### Intended purpose

The product is intended to be used for the therapy and/or prevention of multiple indications, such as phlebological and/or lymphological conditions.

### Indications

- Venous insufficiency
- Oedema
- Lymphoedema
- Deep vein thrombosis
- Post-thrombotic syndrome
- Venous leg ulcers
- Mixed leg ulcers
- Varicose veins
- Direct trauma to the limb, such as from sport-related injuries

### Contraindications

If the following conditions are present, advice from a medical professional is recommended prior to use of compression therapy:

- Peripheral arterial disease (ABPI < 0.8)
- Diabetes
- Presence of cardiac disease
- Severe skin infection
- Peripheral neuropathy
- Other contraindications for compression therapy

### Application instructions

**Important:** It is recommended to apply the bandage first thing in the morning, or after lymph drainage. The product is compatible to use with wound dressings.

It is not necessary to use padding to modify the leg shape - i.e., to make the leg shape more cylindrical. However, to help protect bony prominences or fragile skin, padding or thin tubular bandage can be used.

### Applying the bandage

1. The bandage has a longitudinal, lighter line for the overlap and darker crosslines for the correct stretch. Keep the foot at a 90-degree angle and start the application on the bottom of the foot at the base of the big toe, and wrap toward the outside of the foot first. Place the first darker crossline on top of the foot. To secure the bandage at the base of the toe, overlap the bandage to make a double layer on the first turn. **Important:** To avoid swelling at the toe base, the bandage edge should be placed just below the toes.

2. Wrap the bandage around the foot in a circular fashion. **Important:** Carefully follow the lighter longitudinal lines for a proper overlap, and place the darker crosslines in line with one another on the top of the foot.

3. To secure the heel, four loose turns will be made so that the darker crosslines are positioned on either side of the ankle (see 3a-3d).

3a. For the 1st turn, wrap the bandage so that it fully covers the lower ankle as well as the upper portion of the heel, placing the dark crossline on the inside of the ankle.

3b. For the 2nd turn, position the dark crossline on the outside of the ankle and wrap the bandage under the heel.

3c. For the 3rd turn, position the dark crossline on the inside of the ankle.

3d. For the 4th and final turn, wrap the bandage until the dark crossline is positioned on the outside of the ankle. Continue to wrap the bandage around the ankle.

4. Position the next crossline at the front of the ankle in alignment with the crosslines on the top of the foot. **Important:** Make sure that there are no creases in the bandage material around the area of the heel/ankle.

5. From the ankle, wrap the bandage in circular turns up the leg and position the darker crosslines along the shin. Carefully follow the lighter longitudinal line for the overlap and place the darker crosslines in line with one another along the shin. **Important:** The lighter line should always be visible in order to prevent increased pressure due to additional layers. Stop wrapping the bandage two finger widths below the back of the knee. The end of the bandage is secured with the PressLock® device. See instructions below.

### Applying the PressLock®

6. Apply the PressLock® to the outside of the leg, 10-20 mm below the edge of the bandage (to prevent chafing), with the opening and orange strip facing the bandage roll.

7. Open the PressLock® and place the bandage inside. Cut off any excess material along the orange strip. Remove the orange strip.

8. Close the PressLock®. The bandage is now in place and can easily be reattached for adjustments.

9. To prevent the Lundatex® medical bandage from slipping on the foot, apply surgical tape under and at the sides of the heel, and on the forefoot to secure the first turn of the bandage. A thin nylon sock can be used for both Lundatex® medical and Lundatex® medical cohesive to keep the bandage in place on the foot/ankle.

## ⚠ Warning/precautions

- Remove the product if discomfort or pain occurs. Contact your care provider if the pain is not eliminated or if it gets worse after removal.

- Never increase the layers of bandage material or increase the overlap, as this will increase the pressure and may lead to dangerous constriction.

- Bandage slippage on the foot can lead to dangerous constriction. Please ensure that all steps of the application instructions are carefully followed.

- Do not modify the product yourself in any way, as it might render the device ineffective or could even pose harm to your health.

### Care instructions

The bandage is a single use product. It can be worn for up to 7 days.

The PressLock® fastening device is re-useable and washable. See the care symbols for instructions.

- Please use a washing bag when laundering the PressLock® to prevent the hook material from damaging other items in the wash.

- Do not use bleaching agents, stain removers or softeners.

### Usage and expected lifetime

- The bandage is intended as a single-patient, single-use product, and can be used for a period of 7 days.

- The PressLock® fastening device is intended as a single-patient, reusable product.

- The recommended length of use for the PressLock® is at least 10 washing cycles, or as long as the adhesion to the bandage is sufficient, provided that proper handling is followed (usage, storage, care, etc.).

### Disposal

When the product is ready for disposal, this should be done according to local regulations. The packaging and brochure are recyclable.

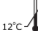
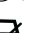
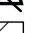
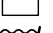
### Report of serious incident

Due to legal regulations, you need to report any serious incident involving the use of this product to the manufacturer, and to your care provider and/or a responsible authority. See contact details to manufacturer on this brochure and on the packaging.

### Warranty

The legal regulations of the country of purchase apply. Please contact the retailer in the event of a potential claim under the warranty. Warranty may be limited or excluded if the handling and care instructions have not been followed. There will be no warranty if the product was not used according to the listed indications, if instructions given by a specialist were not observed, or if the product was modified by the user.

### Symbols

	Expiration date		Keep away from sunlight
	Keep dry		Storage temperature limits
	Consult the Instructions For Use		Caution/precaution
	Medical device		Does not contain natural latex
	Do not dry clean		Do not bleach
	Do not iron		Do not tumble dry
	Dry in shade		Single use only
	Maximum temperature, gentle machine wash		

## Lundatex® medical / *Lundatex*® medical *cohesive*

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